

Accessories List

WARNING: Use only cables, parts and accessories specified or approved by Philips to ensure patient safety. Do not connect any unapproved devices to the MR Patient Care Portal 5000 system. The use and/or connection of parts and accessories not specified by Philips may negatively impact the system and patient safety.

NOTE: Not all accessories and supplies are available in all countries. The list of accessories is subject to change without notice. Contact your Philips sales representatives for other accessories that may be available.

Table 57 Approved Supplies and Accessories

| Description | Part Number |
|--|--------------|
| Compatible Devices | |
| Expression Model MR400 MRI Patient Monitoring System | 866185 |
| Expression MR200 MRI Patient Monitoring System | 866120 |
| Communication Components | |
| Radio Kit, USB | 989803205001 |
| Cradle, Radio | 989803204881 |
| Antenna, Modified HE61 NH, RoHS (Mast Antenna) | 453564892281 |
| Control Room Flex Antenna | 989803176511 |
| Control Room Flex Antenna, RED | 989803206531 |
| Advanced Communications Option | 989803176521 |
| Wall Mounting Accessories | |
| Arm Extension, Strip Chart Printer | 989803181941 |
| Wall Channel, Polymount, 19 | 989803152951 |
| Wall Arm, 16 | 989803179291 |
| Mount, Remote Desktop Wall | 989803204921 |
| Data Input/Output Accessories | |
| USB Flash Drive, Partitioned, | 453564658751 |
| Keyboard and Mouse, Wireless | 989803178091 |
| Bar Code Scanner, Wireless MRI | 989803192381 |
| Thermal Printer Kit, USB | 989803205011 |
| System Power Cords | |
| Cord, Jumper, 25 Feet | 989803168221 |
| Pigtail Line Cord | 453564177511 |
| European Line Cord | 453564177501 |
| UK Line Cord, 3 Meters | 989803174171 |
| North American Line Cord | 989803168211 |

Table 57 Approved Supplies and Accessories (Continued)

| Description | Part Number |
|---------------------------------|--------------|
| Brazilian Power Cord, 3 Meters | 989803173901 |
| Power Cord, S Africa, 3 Meters | 989803181321 |
| Power Cord, AUS/NZL, 3 Meters | 989803181291 |
| Power Cord, Danish, 3 Meters | 989803181331 |
| Power Cord, Israeli, 3 Meters | 989803181341 |
| Power Cord, Argentina, 3 Meters | 989803181351 |
| Power Cord, Swiss, 3 Meters | 989803181361 |

MR Compatibility of System Components

Table 58 lists the Compatibility of the MR Patient Care Portal 5000 components and accessories.

Table 58 MR Compatibility of System Components







| Part Number | Description | MR Rating | Warning |
|--------------|---------------------------------|---|--|
| 453564658751 | USB Flash Drive, Partitioned |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 453564792561 | Desktop Unit |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 989803204941 | Display, 18.5 inch |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 989803205741 | Power Adapter, Desktop |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 453564875441 | Power Supply, Desktop, XP, RoHS |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 989803176511 | Control Room Flex Antenna |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |

Table 58 MR Compatibility of System Components (Continued)









| Part Number | Description | MR Rating | Warning |
|--------------|---|---|---|
| 989803206531 | Control Room Flex Antenna, RED |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 989803178091 | Wireless Keyboard and Mouse, | N/A - OEM | N/A |
| 989803192381 | Bar Code Scanner, Wireless MRI |  Bar Code Scanner Base – MR unsafe  Barcode Hand-held scanner – MR Conditional | Base - Projectile Hazard. Keep out of MR Scanner Room Scanner - The hand-held component of the bar code scanner may be used inside the room with the MR magnet in a location at or outside the 2,000 (2,000 or less) Gauss (0.2T) field line, as measured from the centerline of the MR bore, but in no case closer than 3 feet (1 meter) from the MR system. Keep the hand-held out of the MR system bore and avoid contact with the patient's bare skin while the MRI scan is running. Potential for excessive heating and patient injury. The hand-held is electronic and susceptible to excessive heating from the MRI scan RF only if placed inside the bore of the MR system during that scan. Failure to do so may result in patient injury. |
| 989803205001 | Radio Kit, USB |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 989803204881 | Cradle, Radio |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |
| 989803205011 | Thermal Printer Kit, USB (includes 453564253071 Thermal Printer, USB; and cables) |  MR unsafe | Projectile Hazard. Keep out of MR Scanner Room |

Table 58 MR Compatibility of System Components (Continued)

| Part Number | Description | MR Rating | Warning |
|--------------|------------------------------------|---|---|
| 989803152951 | Wall Channel, Polymount, 19 | N/A - OEM | N/A |
| 989803179291 | Wall Arm, 16 |  MR unsafe (OEM) | Projectile Hazard. Keep out of MR Scanner Room |
| 989803181941 | Arm Extension, Strip Chart Printer | N/A - OEM | N/A |
| 989803204921 | Mount, Remote Desktop Wall | N/A - OEM | N/A |
| 989803176521 | Advanced Communications Option |  MR Conditional | The ACO antenna can be mounted anywhere in the MRI scanner room, except inside the MRI system bore. The device contains non-magnetic metal and is susceptible to excessive heating from the RF produced during an MRI scan if placed inside the MRI system bore. Failure to observe this warning may result in damage to the antenna or patient injury. |